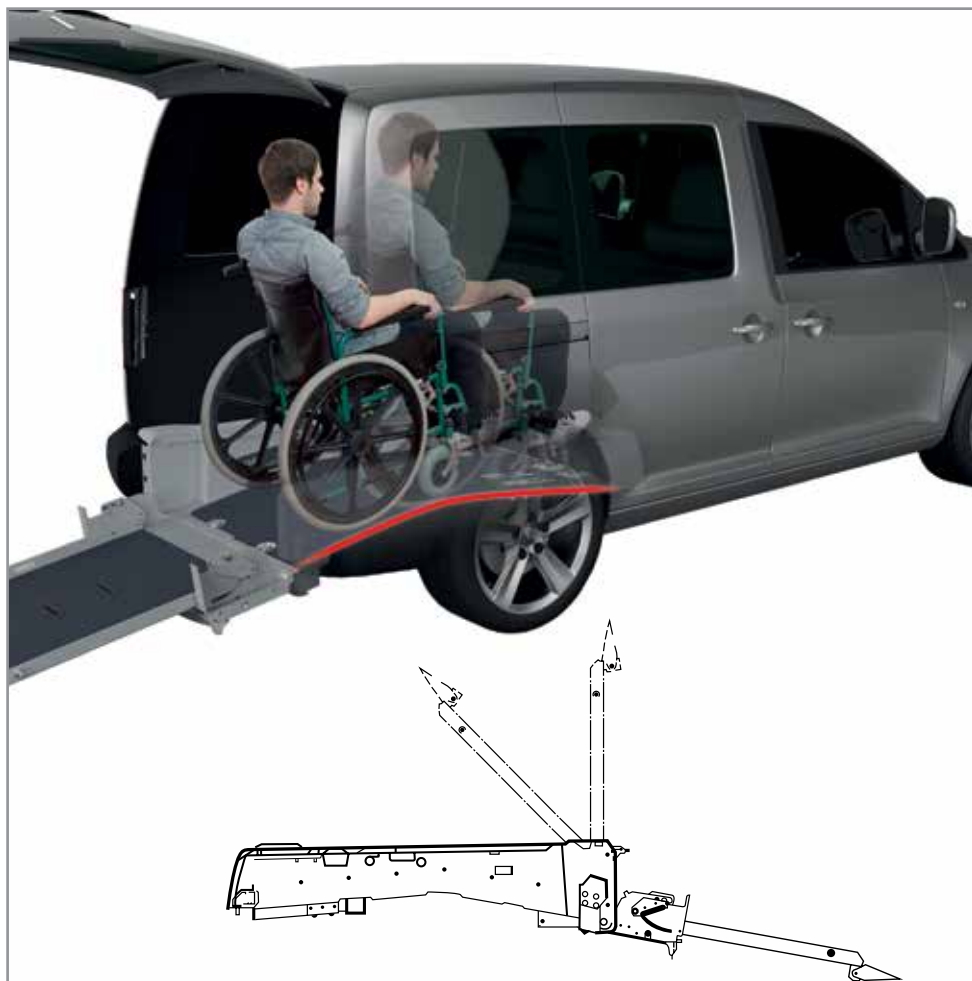
NÁVOD K OBSLUZE / USER INSTRUCTIONS

VW Caddy Flexi Ramp

CZ / EN

VER:

A. ÚVOD / INTRODUCTION

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You must read this instruction manual and thoroughly understand it before using the product. Keep the instructions to hand for future reference. The Flexi-Ramp conversion by API in the VW Caddy Flexi Ramp makes it very simple to transport a disabled person in a wheelchair.

The conversion has been subjected to and has passed all of the relevant safety tests. Any modifications made to the ramp conversion after it has been installed in the vehicle that have not been carried out by or authorized by API will render the warranty invalid and API CZ s.r.o. takes no responsibility for any damage or injury.

When the vehicle is delivered to you, please check the operation of the Flexi-Ramp conversion thoroughly and report any defects immediately.

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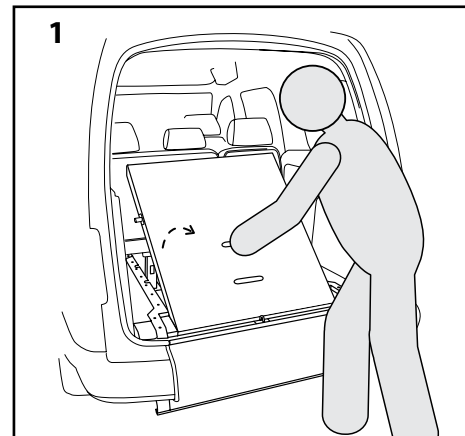
VW Caddy Flexi Ramp is designed to transport a disabled person in a wheelchair. It features a unique folding ramp. Easy manoeuvrability is achieved by good access to the vehicle. The wheelchair is secured in the vehicle using a 4 point retractable restraint system. The wheelchair occupant has a 3 point safety belt that secures them in the wheelchair. This wheelchair restraint system offers the greatest levels of safety for both the wheelchair user and the passengers when driving the vehicle.

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
To ensure safe transportation, always make sure the person is secured with safety belt and wheelchair is tied properly in place, using all 4 points of the wheelchair tie-down system.

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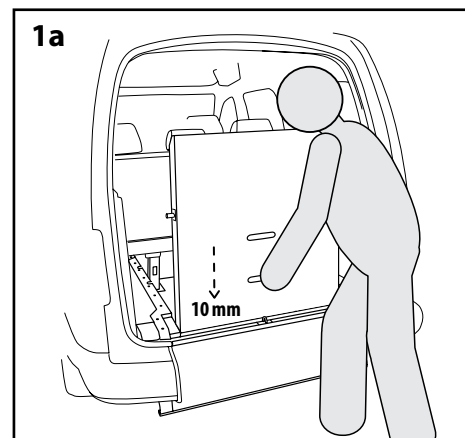
Lift the ramp from the vehicle floor into a vertical position.

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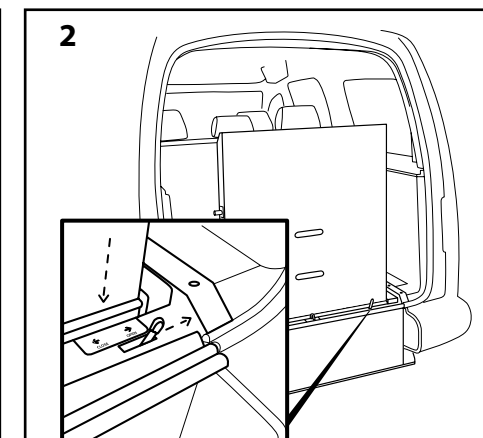
Vertical ramp must go down by about 10 mm. In this position, the ramp is locked against tilting.

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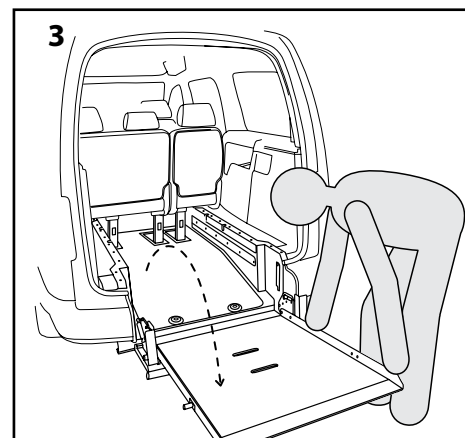Move the lever for opening to the right. This will release the whole ramp for tilting of the vehicle (**Fig. 2**).

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
For folding the ramp back to the vehicle follow the pictures 1–3 in reverse order.

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
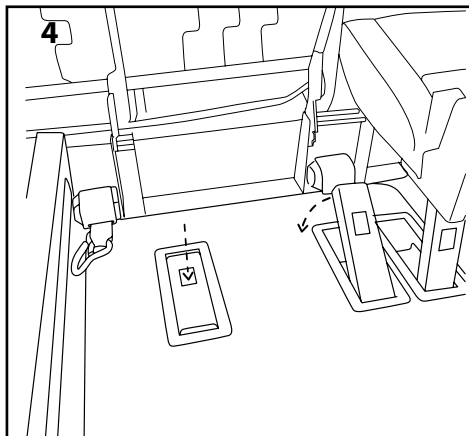
Before loading the wheelchair into the car, lower or remove the rear seats (see the owner instructions of the vehicle) to make a space for the wheelchair. The rear seat legs can be folded flat to the floor once the seat is folded up out of the way. The seat legs can be raised by simply pressing the button in the middle of the leg when it is folded to the floor.

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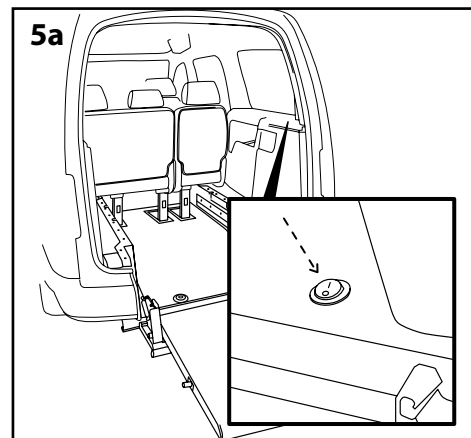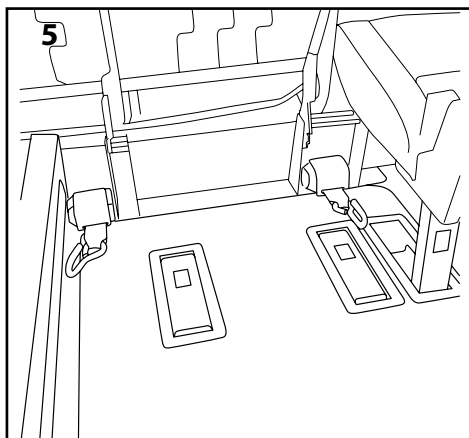
The front retractable belts are placed under the folded seat. To allow the belts to be pulled out, you need to turn on the switch located on the right side at the rear of the vehicle. When the light is switched on the belts are released (Fig. 5a).

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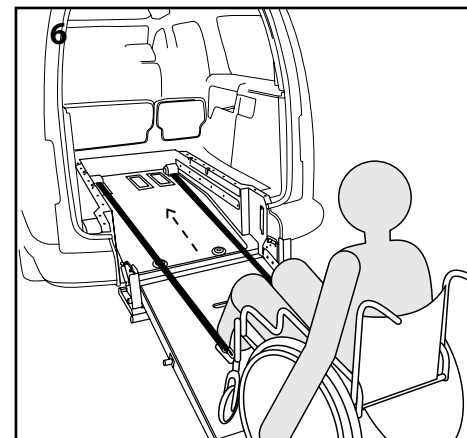Make sure that the switch is switched off when belts are in the vehicle. This will prevent discharging the vehicle battery.

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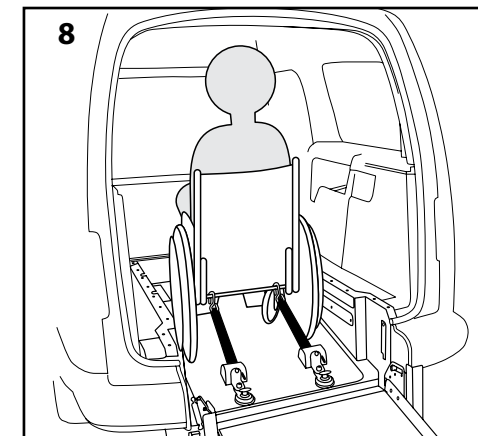
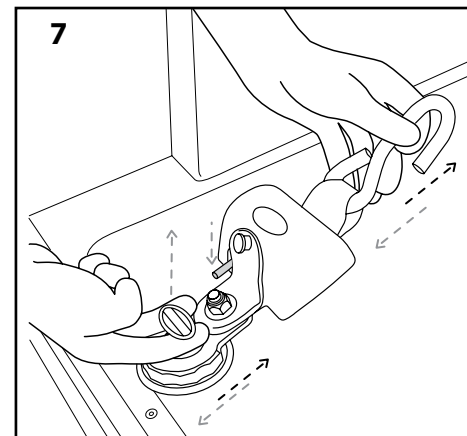
Once the restraints are hooked onto the wheelchair you need to turn the switch off to allow the restraints to lock (switch light turns off). Then carefully push the wheelchair into the vehicle. The restraints will retract into the vehicle and the wheelchair will not be able to roll backwards when loading. If you wish to release the front restraints, you need to turn the switch back on again to allow the restraints to release.

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Do not secure the belts to the wheelchair footrests. Always consult your wheelchair manual and ensure that your wheelchair is manufactured to the required standards to allow it to be used to transport a person in a moving vehicle. You must always secure the restraints to the point on the wheelchair that the wheelchair manufacturer recommends. API CZ s.r.o. will not be held liable for any issues arising from incorrect securement of a wheelchair in the vehicle.

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

Attach the rear retractable belts to the anchorage points in the floor. To unbuckle the belt out of the wheelchair it is necessary to release the red safety lock. The removal is done by pulling the lock up and sliding out of the anchorage point in the floor.

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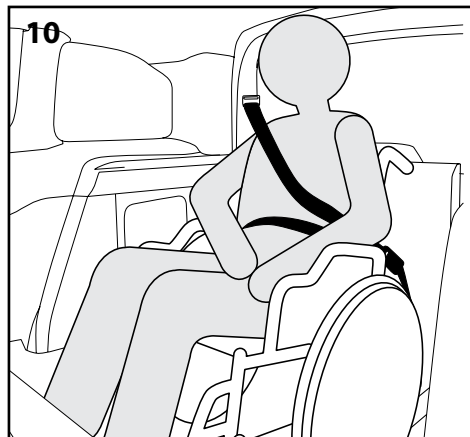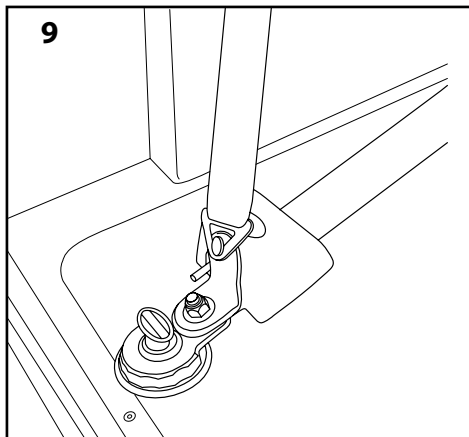Attach rear retractable belts to the wheelchair. (Always consult your wheelchair manual and ensure that your wheelchair is manufactured to the required standards to allow it to be used to transport a person in a moving vehicle. You must always secure the restraints to the point on the wheelchair that the wheelchair manufacturer recommends.)

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Secure the person in a wheelchair by shoulder belt. Close the ramp back to the vehicle.

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Repairs of damages incurred on VW Caddy Flexi Ramp conversion can be carried out only by authorized workshops.

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Keep the ramp latch and the retractable restraints clean and damage free.

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Fuel level gauge

Due to the changed tank, the fuel level display shows the volume in the tank as follows:

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Spare parts are supplied exclusively by the company API CZ s.r.o.

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All information contained in this manual is valid at time of printing. API CZ s.r.o. reserves the right to changes without notice. © API CZ s.r.o 2011

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WARRANTY CONDITIONS:

- The warranty will last the duration of the original vehicle, subject to all of the warranty conditions of the original vehicle being met.
- The warranty covers mechanical parts of the product and its functionality.
- The warranty does not cover defects or malfunctions caused by unprofessional installation, adjustment or repair of the product, product damage due to accidents and defects caused by misuse of the product.
- Every 30,000 km or every 2 years it is necessary to have the vehicle inspected.
- In case the warranty is acknowledged and the guarantee defect is removable, replacement of the product cannot be required.
- The warranty does not include transportation costs. The customer is responsible for transporting the vehicle to and from the supplier for any warranty work to be completed.
- API is not liable for any direct or indirect damages resulting from a vehicle accident or malfunction of the product installed.
- Warranty claims do not remove the necessity for the product to be paid for in full

Datum / Date:

.....